

Saliva Collection Kit Instructions

For In Vitro Diagnostic Use



Read all instructions prior to collection.

COLLECTION PRECAUTIONS:

DO NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.


DO NOT remove the plastic film from the funnel lid.

Intended use: For the collection of viral RNA and human DNA.

Contents: Kit contains stabilizing liquid.

Warnings and precautions: Wash with water if stabilizing liquid comes in contact with eyes or skin. The small cap is a choking hazard. Do not drink the stabilizing liquid.





Storage: 15°C - 30°C

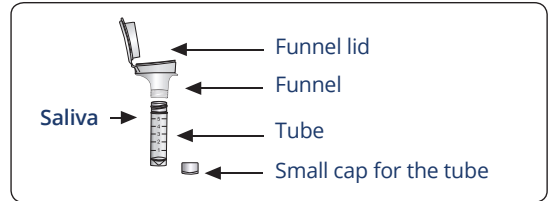

Summary and explanation of the kit: GENETWORx Saliva Kit is a self-collection kit that provides the materials and instructions for collecting and stabilizing saliva specimens.

Procedure: It will likely take between 2 and 5 minutes to deliver a saliva sample that reaches 1ml.

Follow Steps 1 - 7 in order. Read all directions thoroughly and carefully.

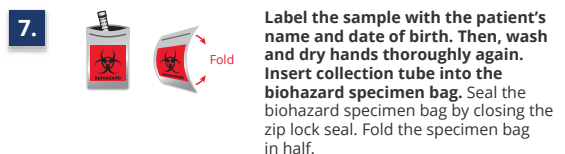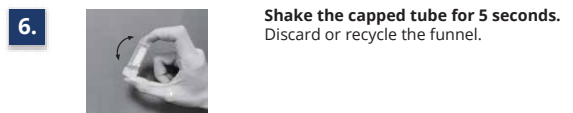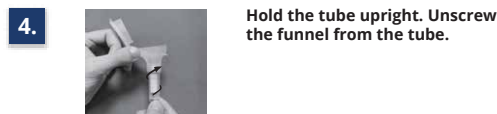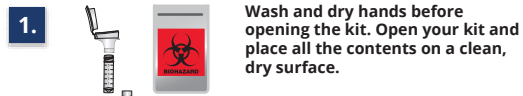

For In Vitro Diagnostic Use

